


To determine proper liner sizing, two measurements are required:

Step #1: Take circumference measurement 2" (5-6cm) proximal to the distal end of the residual limb.

Step #2: Take length measurement from mid-patella tendon to the distal end of the residual limb.

Note: Unless specified, all fabric covered Seal-Pro TT liners come with proximal 3" exposed. If custom fabric is desired, please state your description at the time of order.

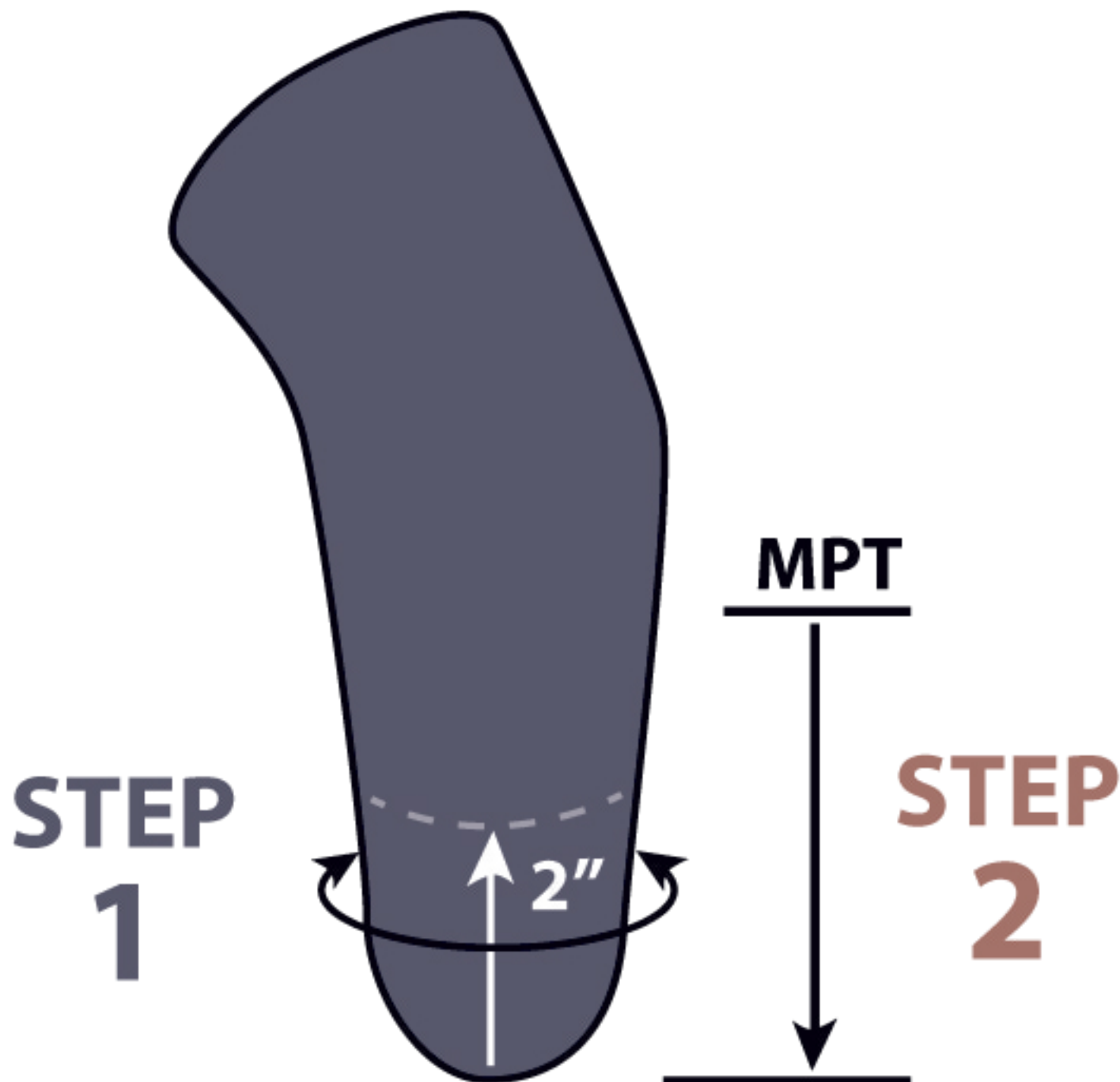

EXAMPLE: SP-M- 6 L Custom

Limb Circ. Of 28" and an MPT to Distal Length of 6" plus Locking with Custom Textile.

STEP 1: Limb Circ.	Choose Part Number
14 - 20 cm	SP-XS - A B
20 - 26 cm	SP-S - A B
26 - 32 cm	SP-M - A B
32 - 38 cm	SP-L - A B
38 - 42 cm	SP-XL - A B

STEP 2: MPT to Distal	Insert # for "A"
4"	4
6"	6
8"	8

STEP 3: Custom Textile Requests	Insert Code for "B"
Cushion NO Textile	(Leave Blank)
Cushion w/ Textile Prox. 3" Exposed	F
Cushion w/ Custom Textile to Spec	F Custom
Locking w/ Textile Prox. 3" Exposed	L
Locking w/ Custom Textile to Spec	L Custom