Instructions for use

Extreme®

Life Without Limitations®

.....	3
ⒺⒺ Instructions for use	5
ⒺⒺ Gebrauchsanweisung	5
ⒺⒺ Notice d'utilisation	6
ⒺⒺ Instrucciones para el uso	7
ⒺⒺ Istruzioni per l'uso	8
ⒺⒺ Bruksanvisning	9
ⒺⒺ Brugsanvisning	10
ⒺⒺ Bruksanvisning	11
ⒺⒺ Käyttöohjeet	12
ⒺⒺ Gebruiksaanwijzing	13
ⒺⒺ Instruções de Utilização	14

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FITTING INSTRUCTIONS

- **Mid-strap (Fig. 1):** Fasten the midstrap by passing it behind the calf, inserting it through the corresponding D-ring on the opposite side of the calf shell and back onto itself. Make sure this is secured snugly, as is the primary suspension strap.
- **Thigh strap (Fig. 2):** The thigh strap passes through the slotted thigh strap attachment and doubles back onto itself.
- **Dynamic strap:** The D.F.S. spirals on a diagonal around the back of the leg and threads through the D-ring on the front of the thigh shell, then fastens back onto itself (**Fig. 3**). The calf strap passes through the lower D-ring and doubles back onto itself (**Fig. 4**). Finally, bend the knee to 80° and retighten the D.F.S. When the D.F.S. is adjusted correctly, it tightens each time the leg approaches extension (**Fig. 5**). Secure any accessory straps that may have been added to improve brace suspension and/or ligament support.

BRACE MAINTENANCE

Your Ossur knee brace is designed for minimum maintenance and care. Follow these easy steps to keep your knee brace in top condition.

- **Hinge Lubrication:** A tube of heat resistant silicone lubricant is supplied with each Ossur brace. Apply a small quantity with a toothpick or similar device to the inner surfaces of the hinge, where visible. Lubricate the hinge several times a year. **DO NOT DISASSEMBLE THE HINGE!**
- **Liner Cleaning:** All liners may be hand washed using a sponge or soft cloth and cold water with mild detergent. Rinse thoroughly and air dry.
- **Strap Cleaning:** Straps may be hand washed using a mild detergent soap and a soft brush, then rinsed thoroughly and air-dried. **DO NOT REMOVE STRAPS.**
- **Reconditioning:** Wear and time may take their toll on your brace. Your brace can be fully reconditioned.

BRACE ACCESSORIES

Össur offers you a line of accessories that are designed to meet the specific needs of you Össur brace. These accessories can be easily purchased through your brace provider or the Ossur Customer Service Department.

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Össur Iberia S.A.