





Instructions for Use

FOOT COVER

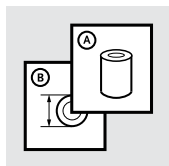


		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	9
ES	Instrucciones para el uso	11
IT	Istruzioni per l'uso	13
NO	Bruksanvisning	15
DA	Brugsanvisning	17
SV	Bruksanvisning	19
EL	Οδηγίες χρήσης	21
FI	Käyttöohjeet	23
NL	Gebruiksaanwijzing	25
PT	Instruções de Utilização	27
PL	Instrukcja użytkowania	29
CS	Návod k použití	31
TR	Kullanım Talimatları	33
RU	Инструкция по использованию	35
JA	取扱説明書	37
ZH	中文说明书	39
KO	사용 설명서	41

A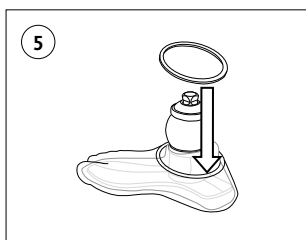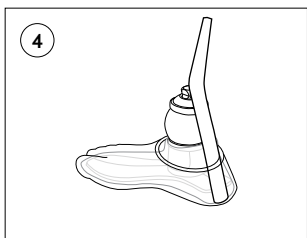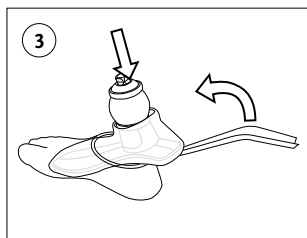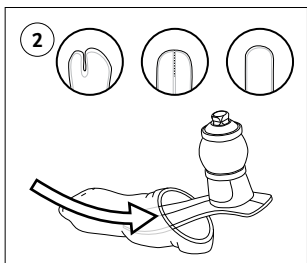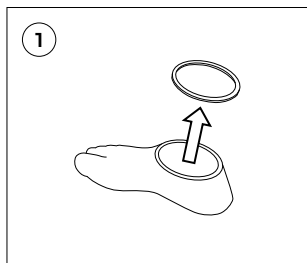

B

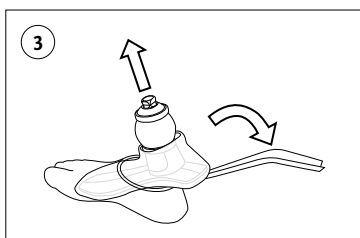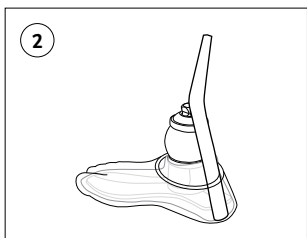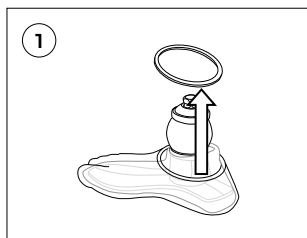	Sizing guide table			
Foot cover (cm)	US (M)	US (W)	Euro	UK
22	3.5 – 4.5	5 – 6	35 – 36	2.5 – 3.5
23	5 – 5.5	6.5 – 7	37	4 – 4.5
24	6 – 7	7.5 – 8.5	38 – 39	5 – 6
25	7.5 – 8	9 – 9.5	40	6.5 – 7
26	8.5 – 9	10 – 10.5	41 – 42	7.5 – 8
27	9.5 – 10	11 – 12	43	8.5 – 9.5
28	11 – 11.5	12.5 – 13	44 – 45	10 – 10.5
29	12 – 12.5		46	11 – 11.5
30	13		47	12

ENGLISH

The foot cover is referred to as the **device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for a certified prosthetist and the user of the device.

PRODUCT DESCRIPTION

The device is a cosmetic cover intended to securely contain the intended Össur prosthetic foot, providing an interface between the foot and a shoe or the ground as part of a prosthetic system.

THIS KIT INCLUDES

- Foot cover
- Attachment plate
- Spectra sock (black)

INDICATIONS FOR USE

Lower limb amputations.

CONTRAINDICATIONS FOR USE

None known

INTENDED USE

User activity levels: Low-High.

Foot size: 22-30 cm (see sizing guide table).

For FSF: Foot size 22-26 cm.

ASSEMBLY

Inserting a prosthetic foot into the device:

- Remove the attachment plate from the device opening (**Figure A1**).
N.B. FSF footcover does not have an attachment plate.
- Place the Spectra sock over the prosthetic foot in one of the following ways, depending on foot type (**Figure A2**):
 - Form the sock in between the sandal toes.
 - Form the sock stitching between the carbon blades.
 - Place the sock over the foot.**N.B.** When donning the Spectra sock, ensure the stitching in the toe area is facing upwards.
- Using the Össur shoe horn as shown, insert the foot into the device (**Figure A3**). Ensure the carbon fiber heel snaps in securely under the heel clip in the posterior part of the device (**Figure A4**).
- Reattach the attachment plate to the opening of the device (**Figure A5**). Start by engaging the posterior pins, progressing forward, finally engaging the pins at the anterior part of the opening.
N.B. FST foot cover: ensure the arrow on the attachment plate is pointing anteriorly.

Removing the device:

- Remove the attachment plate from the opening of the device (**Figure B1**).
- Insert the Össur shoe horn under the carbon fiber heel at the posterior end, and pry the foot out of the heel clip (**Figure B2**).
- Pull the foot out of the device (**Figure B3**).

N.B. Avoid frequent removal and insertion of the prosthetic foot into the device, as this can cause damage to the carbon fiber elements and/or the device.

N.B. Do not expose the device to temperatures of greater than 50°C for prolonged periods of time.

MAINTENANCE

Regular cleaning of the device exterior using hygienic wipes is recommended. For stubborn stains use Isopropyl Alcohol.

In cases of damage or changes in device performance, the user should immediately cease using the device, and consult their practitioner.

WARRANTY

Össur provides a 6-month (from invoice date) limited warranty on the device.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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