


GENU-RANGER® Range of Motion Knee Brace


Application Instructions

Universal Size Fits Either Left or Right Leg

Reorder No.	Size	Description
0814 1603	Short	28" Inseam or less
0814 1600	Regular	28" - 32" Inseam
0814 1601	Long	32" Inseam or more

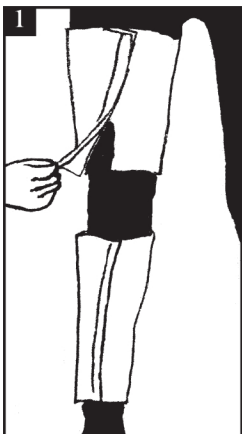

1. Position leg wrap closures in front of patient to allow for one piece brace removal - Trim if necessary.

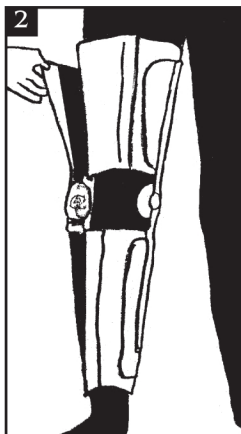

2. Position hinge/struts medial and lateral with alignment slightly superior and posterior of the patella center point.

3. Attach circumferential straps beginning with straps immediately above and below the patella.

Position the remaining calf and thigh straps and fasten securely.



4. To set range of motion limits, remove the Allen Nut using the allen wrench provided. Turn cap to expose pins. Dial-in correct position, and Allen Nut to obtain added security against noncompliance.



CAUTION

Both hinges should have similar range of motion settings. This brace should not be removed or hinge settings adjusted without specific instruction from a physician. Federal law restricts the device to sale by or on the order of a physician or duly licensed practitioner.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com

©2005 Bird & Cronin, Inc. All Rights Reserved - 10001600