

# GENU TRAC™ KNEE STRAP

*Designed To Help Relieve Pain Associated with Arthritis,  
Tendonitis and Chondromalacia*

**LATEX FREE**



## Application Instructions

1. Wrap strap with plush side out just underneath the patella and fasten inside hook tab to the plush side on the medial or lateral side of the knee.
2. Position the buttress underneath the patella and secure with hook tab on back of the buttress.
3. Wrap the remaining length of strap around the patella area and secure with hook tab on end of strap.
4. Strap may be trimmed after initial fitting if desired for less bulk. Be sure to trim the end without the hook tab, and always position the inside tab to the medial or lateral side of the knee.

Reorder Number	Size	Unit
5000 0140	Universal	Each

***BIRD & CRONIN INC.***

*Designers and Manufacturers of Quality Medical Products*

1200 Trapp Road • Eagan, Minnesota USA 55121

800.328.1095 • Fax 800.279.7934

email sales@birdcronin.com • www.birdcronin.com

10000140