

## Lumbosacral Support - Tapered Style



### Application Instructions

1. Place support on lumbosacral region so the four metal stays are positioned in back. The contoured mesh back should be facing up, as shown above.
2. Wrap around abdomen securing with hook and loop closure.
3. Fasten buckle straps and adjust for comfort.
4. Re-adjust if necessary.

Reorder No.	Size	Hip Measurement	CM
0814 4331	X-Small	24" - 28"	61. - 71.
0814 4332	Small	28" - 32"	71. - 81.
0814 4333	Medium	32" - 36"	81. - 91
0814 4334	Large	36" - 40"	91. - 102
0814 4335	X-Large	40" - 44"	102. - 112
0814 4336	XX-Large	44" - 48"	112 - 122
0814 4337	XXX-Large	48" - 52"	122 - 132

## ***BIRD & CRONIN INC.***

*Designers and Manufacturers of Quality Medical Products*

1200 Trapp Road • Eagan, Minnesota USA 55121  
651.683.1111 • Fax 651.688.9855 • 800.328.1095 • Fax 800.279.7934  
e-mail sales@birdcronin.com • www.birdcronin.com  
10004331