



BIRD & CRONIN

PRIMO WRIST BRACE

Application Instructions



1. Measure the wrist circumference to determine the proper size. Slide brace on the affected hand with the layer of stockinette on the back of the hand.
2. Thread hook and loop through D-rings securing the hook and loop closures. The metal stay should fit into the palm of the hand and extend down the forearm.
3. Trim the straps leaving approximately 1" - 2" of overlap.

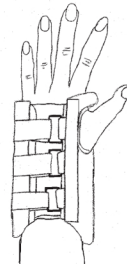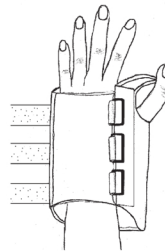

WASHING INSTRUCTIONS

Remove metal stay. Fasten hook and loop. Hand wash in warm water using a mild detergent. Air dry. Blot excess water with a towel, do not wring out.

Reorder No.	Reorder No.	Wrist
Left	Right	Circumference
0814 7391	0814 7381	X-Small (5 3/4" - 6 1/2")(16.5cm)
0814 7392	0814 7382	Small (6 1/2" - 7 1/4")(16.5cm-18.4cm)
0814 7393	0814 7383	Medium (7 1/4" - 8 1/4")(18.4-21cm)
0814 7394	0814 7384	Large (8 1/4" - 9")(21-22.9cm)
0814 7395	0814 7385	X-Large (9" - 10")(22.9-25.4cm)

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

800/328-1095 • Fax 800/278-7934 • 651/683-1111 • Fax 651/688-9855

e-mail sales@birdcronin.com • www.birdcronin.com

©2009 Bird & Cronin, Inc. All Rights Reserved -1000 7391