



BIRD & CRONIN INC.

RX 8 Universal Wrist Brace

APPLICATION INSTRUCTIONS

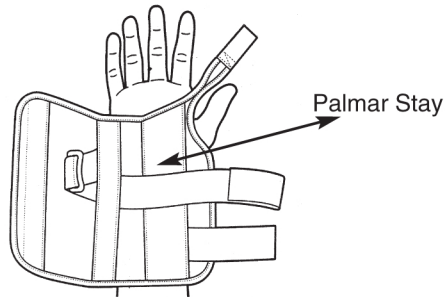

1. Loosen closures. Slip the splint onto the affected hand.
2. Position the dorsal stay on top of the hand in opposition to the palmar stay.

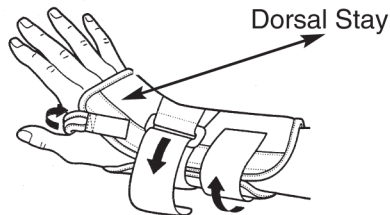

3. Pass circumferential wrist strap through the loop lock closure. Secure the remaining closures and adjust for comfort.

REORDER NUMBER

Left

0814 4241

REORDER NUMBER

Right

0814 4240

SIZE

Universal

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com

©2010 Bird & Cronin, Inc. All Rights Reserved - 10004240