

T.P.C. Cervical Collar with Trachea Opening

Application Instructions



1. Please select the correct size as the collar comes in different heights and circumferences. Apply collar piece "Back" to the back of patient's neck. Height - measure bottom of chin to top of sternum.



2. Apply front half of collar with chin secured in recess. Center collar to ensure neutral alignment. Front half overlaps back half.



3. With the hook and loop fasteners, tighten collar with bi-lateral adjustments. Center collar with arrow pointing up.



4. Adjust if necessary.

Height	2.25"	3.25"	4.25"	5.25"
Small Cir.: 10" - 13"				
Medium Cir.: 13" - 16"				
Large Cir.: 16" - 19"				
X-Large Cir.: 19" - up				

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121
 800/328-1095 • Fax 800/279-7934 • 651/683-1111 • Fax 651/688-9855
 email sales@birdcronin.com www.birdcronin.com 10007142