


Soft Cervical Collar


APPLICATION INSTRUCTIONS

CP020401 Rev C 08/10 L


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

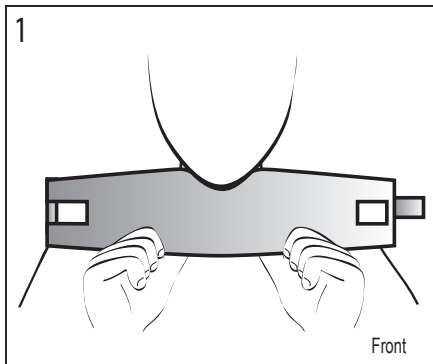

1. Position Collar
Center the cervical collar under the patient's chin.

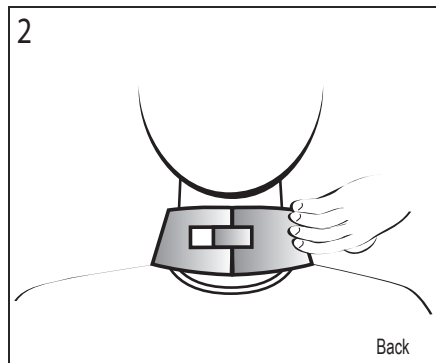

2. Secure Collar
Wrap both ends around the back of the neck and secure hook and loop closure.

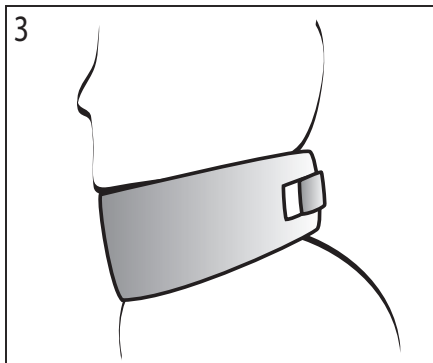

3. Adjust Collar
Adjust for comfort and ensure collar does not interfere with normal breathing.



Council Directive 93/42/EEC
of 14 June 1993 concerning Medical devices

INDICATIONS: Indicated for Cervical Facet Syndrome, discogenic pain, Degenerative Disc Disease and muscle sprains or strains of the cervical region.

CONTRAINDICATIONS: Contraindicated for unstable fractures of the neck.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1.888.BLED SOE (253.3763).

MANUFACTURED BY:
Medical Technology, Inc. d/b/a
BLED SOE BRACE SYSTEMS™
2601 Pinewood; Grand Prairie, Texas, 75051, USA
Toll Free Tel: 1.888.BLED SOE (253.3763) • 1.800.527.3666 Local:
972.647.0884 • Local Fax: 972.660.5495
www.BledsoeBrace.com
© 2010 Medical Technology, Inc. All Rights Reserved

EUROPEAN AUTHORIZED REPRESENTATIVE

| | | |
|----|-----|--|
| EC | REP | EMERGO EUROPE Molenstraat 15 2513 BH, The Hague The Netherlands Tel: (+31) 70 345 8570 Fax: (+31) 70 346 7299 |
|----|-----|--|