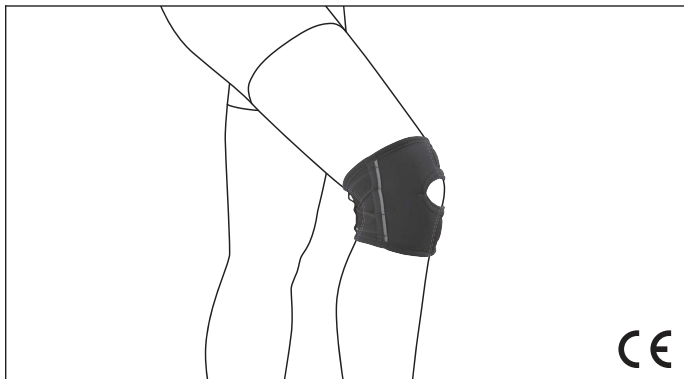
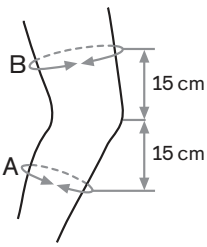
8360 Genu Carezza Patella Stabilizer

DE	Gebruiksaanweisung	3
EN	Instructions for use	7
FR	Instructions d'utilisation	11
IT	Istruzioni per l'uso	15
ES	Instrucciones de uso	19
NL	Gebruiksaanwijzing	24
SV	Bruksanvisning	28
DA	Brugsanvisning	32
NO	Bruksanvisning	36
FI	Käyttöohje	40
PL	Instrukcja użytkowania	44
HU	Használati utasítás	48
CS	Návod k použití	52
HR	Upute za uporabu	56
SK	Návod na používanie	60
BG	Инструкция за употреба	64
TR	Kullanma talimatı	68
RU	Руководство по применению	72
ZH	使用说明书	77

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INFORMATION

Date of last update: 2020-04-29

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the fitting and application of the 8360 Genu Carezza patella stabilizer knee support.

2 Intended use

2.1 Indications for use

The support is intended **exclusively** for fittings of the lower limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

2.2 Indications

- Patellofemoral pain syndrome (e.g. chondromalacia patellae, chondropathia patellae, retropatellar osteoarthritis)
- Patellofemoral maltracking
- Following patellar dislocation/subluxation

- After lateral release operations

The indication must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

None known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the medical device will be applied; sensory and circulatory disorders in the leg area.

2.4 Mechanism of Action

The support stabilizes the kneecap laterally and relieves pain.

3 Safety

3.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
---	---

 NOTICE	Warning regarding possible technical damage.
--	--

3.2 General safety instructions

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- The product may be used by one person only.
- Clean the product regularly.

NOTICE**Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

NOTICE**Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Selecting the Size

- 1) Measure the circumference of the thigh 15 cm above the center of the patella.
- 2) Measure the circumference of the lower leg 15 cm below the center of the patella.
- 3) Determine the size of the support (see size chart).

4.2 Fitting and Application

⚠ CAUTION**Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.

- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- > The patient is sitting on the front edge of a chair.
- > Open all hook-and-loop closures on the support.
- 1) Hold the support by the top edge on both sides and pull it over the knee until the patella cutout is centered on the patella.
- 2) Fasten the straps first below and then above the patella (see fig. 1).

4.3 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the support regularly:

- 1) Fasten all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.
- 3) Wash the support in warm water at **40 °C (104 °F)** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

6.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

6.2 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

6.3 CE Conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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