
ICELOCK® VALVE 641

Life Without Limitations®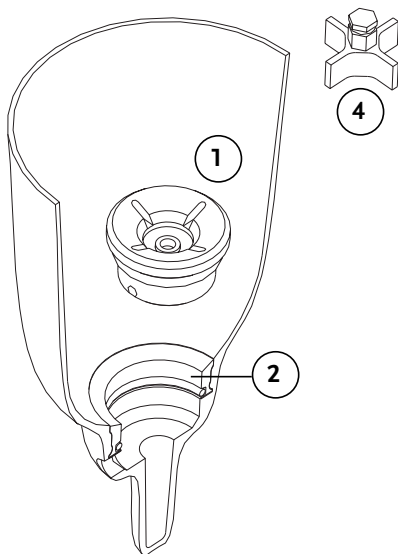

ENGLISH

ICELOCK® VALVE 641

L-641000

CONTENT

L-692022 Fabrication Ring w/ o-ring & silicone ring **(Figure 2)**.

L-692116 Expulsion valve, silicone, **(Figure 1)**.

L-693112 Silicone Seal **(Figure 3)**.

L-693125 Replacement o-ring.

Items 2,3 & Replacement o-ring are used when the socket is fabricated.

Use with L-680000 Fabrication Tooling (reusable).

TOOLS REQUIRED

- 8mm (5/16) socket wrench.

INSTALLATION

- Following socket manufacture, replace existing o-ring with replacement o-ring to provide uncontaminated seal between Fabrication ring and Attachment.
- Screw Bottom Attachment w/ funnel and pre-inserted silicone Expulsion Valve **(Figure 1)** into bottom of socket.
- Torque to 8Nm (6ft.lb) using Cross Driver **(Figure 4)** (comes with Fabrication Tooling) and a 8mm torque wrench.
- Don socket and seal brim with an ICEFLEX suction suspension sleeve.

IMPORTANT

To prevent leakage, fasten the socket in a vice and torque the valve to 8 Nm (6ft.lb.) using the Cross Driver and 8mm (5/16) torque wrench.

CLEANING

It is recommended that Expulsion Valve **(Figure 1)** be cleaned at least every six weeks to ensure proper functioning. Clean with air and / or water & soap.

Distal/weight bearing component determines weight limit.

Other locking components available for 600 series

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L-631000 Icelock Lanyard 631

L-651000 Icelock Smooth 651

EN – Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If unusual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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Össur Iberia S.A.