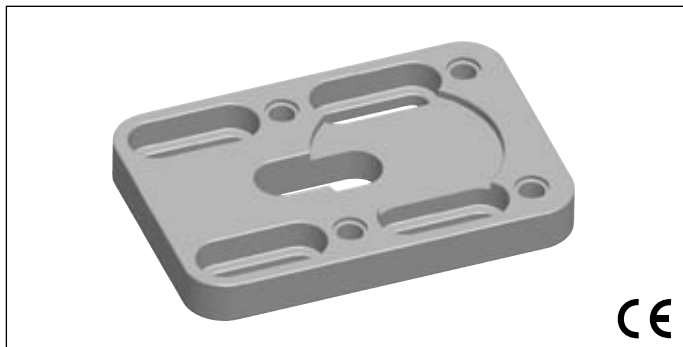
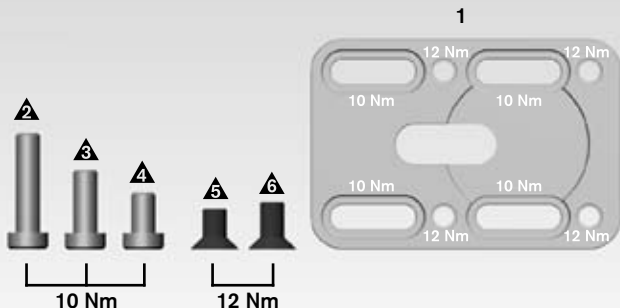
4R118

DE	Gebrauchsanweisung	4
EN	Instructions for Use	8
FR	Instructions d'utilisation	12
IT	Istruzioni per l'uso	16
ES	Instrucciones de uso	19
PT	Manual de utilização	24
NL	Gebruiksaanwijzing	27
SV	Bruksanvisning	32
DA	Brugsanvisning	36
FI	Käyttöohje	39
PL	Instrukcja użytkowania	43
HU	Használati utasítás	47
CS	Návod k použití	51
RO	Instrucțiuni de utilizare	54
TR	Kullanma talimatı	58
EL	Οδηγίες χρήσης	62
RU	Руководство по применению	66
JA	取扱説明書	70
ZH	使用说明书	74

4R118

1

●	▲	■
(D) Einzelteile-Pack	Mindestmenge	Einzelteile
(GB) Single-Component Pack	Minimum order quantity	Single components
(F) Kit de pièces de rechange	Quantité minimum	Pièces à l'unité
(I) Imballo dei singoli componenti	Quantità minima	Singoli componenti
(E) Kit componentes	Cantidad mínima	Componentes
(P) Conjunto de peças de reposo	Quantidade mínima	Componentes
(NL) Onderdelenpakket	Minimikvantitet	Onderdelen
(S) Service-Set	Minimum aantal	Separata delar
(DK) Komponentpakke	Mindste mængde	Komponenter
(PL) Zestaw naprawczy	Ilość minimalna	Podzespoły
(H) Alkatrész-csomag	Minimum mennyiség	Alkatrészek
(CZ) Sada jednotlivých dílů	Minimální množství pro objednání	Jednotlivé díly
(RO) Set componente individuale	Canitate minimă	Componente individuale
(HR) Pakiranje pojedinačnih dijelova	Minimalna količina	Pojedinačni dijelovi
(TR) Müfrit parça ambalajı	Minimum miktar	Münferit parçalar
(GR) Σετ μεμονωμένων εξαρτημάτων	Ελάχιστη ποσότητα	Μεμονωμένα εξαρτήματα
(RUS) Комплект деталей	Минимальное количество	Отдельные детали
(J) 交換パーツセット	最低注文個数あり	個別販売
(中) 维修组件	最低起订量	单个部件

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Date of the last update: 2013-02-11

- Please read this document carefully.
- Follow the safety instructions.

1 Scope of Delivery (Fig. 1)

1.1 *Single-Component Pack* ●

none

1.2 *Minimum order quantity* ▲

- (2) 501T61=M6×30 Cap Screws
- (3) 501T61=M6×25 Cap Screws
- (4) 501T61=M6×12 Cap Screws
- (5) 501S41=M6×12 Countersunk Head Screws (hexagon socket)
- (6) 501S41=M6×16 Countersunk Head Screws (hexagon socket)

1.3 *Single components* ■

none

2 Description

2.1 *Intended use*

The 4R118 Adapter Plate is to be used **solely** for the prosthetic fitting of lower limb amputees.

2.2 Field of application

Field of application according to the **Ottobock MOBIS Mobility System**:

Approved for a **patient weight of up to 125 kg (275 lbs)**.

**Caution!**

Avoid exposing prosthetic components to corrosive elements such as fresh water, salt water, acids and other liquids. Using this medical device in such conditions will render all claims against Otto Bock HealthCare null and void.

Please inform your patients of this caution as well!

The 4R118 Ottobock Adapter Plate is used to optimize the alignment of modular lower limb prostheses. In particular, it serves for the permanent additional posterior placement of the knee joint in relation to the prosthetic socket.

**Caution!**

The Adapter Plate may only be installed between the prosthetic socket and the knee joint.

It must only be used in combination with an adapter with a four-hole connection (5R1=1; 2; 6; 5R2 and 5R6=1; 2 ;3) (proximal). An attachment block with pyramid receiver (4R55, 4R95, 4R22, 4R51, 4R37) is to be used for the distal connection.

2.3 Function

The Adapter Plate allows for a displacement from 10 mm (Fig. 2) up to 25 mm (Fig. 3).

3 Technical Data

Article number	4R118
Weight	75 g
System height	10 mm
Material	Aluminium
Maximum patient weight	125 kg
Mobility level	–

4 Handling

4.1 Handling

Caution!

The Adapter Plate must always be mounted to the adapter with four-hole connection so as to have full contact. Therefore, fix the Adapter Plate by evenly tightening the attachment screws in diagonal order.

As specified in the respective instructions for use, the laminate edges of the 5R1 Socket Attachment Block and the 5R2 Lamination Disk are not to be removed.

However, if using the 4R118 Adapter Plate, the laminate edge can be sanded down to the socket attachment block or the lamination disk to enable mounting the adapter plate.

1. Mount the Adapter Plate to the four-hole connection with the desired posterior placement using the cap screws (501T61=M6x*).
2. Mount the adapter with socket adapter to the 4R118 Adapter Plate using the countersunk head screws (501S41=M6x*). Tighten the countersunk head screws (501S41=M6x*) to 12 Nm and the cap screws (501T61=M6x*) to 10 Nm. Use a 710D4 Torque Wrench.

Caution!

When screwing the 4R118 Adapter Plate to the socket adapters:

5R2 → 501T61=M6x12

5R1=1, 5R1=2, 5R1=6 → 501T61=M6x25

5R6=1, 5R6=2, 5R6=3 → 501T61=M6x30

use only the enclosed countersunk head screws.

Make sure that the screws are screwed to minimum thread depth of **8 mm** to avoid stress fractures!

Attention!

Only the 501S41=M6x16 Countersunk Head Screws provided may be used for screwing the adapter plate to the socket adapters.

When mounting the 4 countersunk head screws, tighten the screws in the following order:

1. Tighten both countersunk head screws on the displacement (posterior) side with controlled torque to **12 Nm**.
2. Afterwards, tighten the two countersunk head screws on the opposite side with controlled torque to **12 Nm**.

Make sure that the screws are screwed to minimum thread depth of **8 mm** to avoid stress fractures!

4.2 Maintenance Instructions

Note:

As a basic principle, all Ottobock modular adapters are subjected to tests involving three million load cycles. Depending on the amputee's activity this corresponds to a service life of three to five years.

We recommend carrying out regular safety checks once a year.

5 Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the Instructions for Use. The device must only be operated with tested modular components in accordance with the Ottobock Mobility System. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

6 Warranty

Ottobock warrants this product/these products from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction. Because warranty conditions may differ by country, please contact your local Ottobock distribution company for more information.

7 CE Conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock HealthCare GmbH